


# QUICK USER GUIDE - MindMe Pro LS

## Starter Guide

### MindMe Pro LS Overview

The MindMe Pro LS (MMPLS) combines a mobile phone, a GPS device, and GPRS satellite technologies to assist in tracking and monitoring vulnerable individuals in one easy to use solution

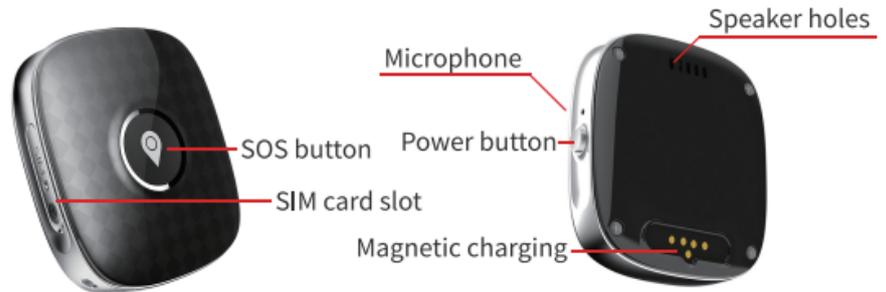

### Turning ON MMPLS


Press and hold down the On/Off button. The MMPLS will vibrate and sound with tune



#### NOTE

- The MMPLS may take up to 30 secs to boot up

### Charging

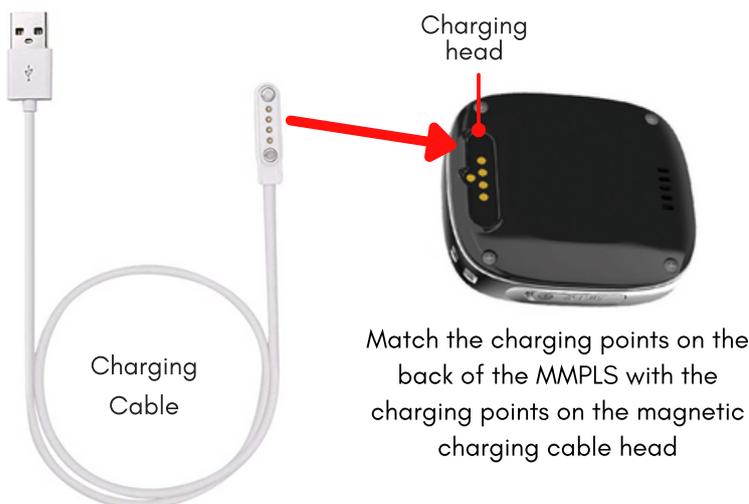

#### NOTE

- When charging, LED flashes periodically, once fully charged the LED light remains on
- Before using your MMPLS please charge the watch for 3-4 hours.
- After charged to 100% the battery will last up to 24 - 48 hours.
- When the battery charge falls below 20% the MMPLS will SMS an alert to the primary number on your MMPLS.



It is a good idea to charge the MMPLS next to the wearer's bed. That way when the wearer goes to bed the MMPLS can be placed on the charger and be fully charged, ready to go when the wearer gets up in the morning. This method also helps in create a routine for the wearer as well as the carer.

# QUICK USER GUIDE - MindMe Pro LS

## Starter Guide

### Using the MMPLS

#### Emergency Procedure



Press and hold SOS button for 7 seconds. The LED light will flash for 3 secs and an audible ring will sound. This initiates the emergency procedure

Alert SMS is sent to the primary emergency contact (including the date, time, google maps location of the MMPLS). Also sends an alert to the mobile APP

Call cycle begins. First SOS contact # is called. If call is unanswered, the call moves to the next SOS contact # in sequence of order within 12 secs

#### NOTE

- Emergency contacts can be either mobiles and landlines. Only the primary number with a mobile phone will receive an SMS with the alerts and a Google Maps link. All emergency contacts will receive an alert via the mobile APP (note: Emergency contact needs to be logged into mobile APP to receive alerts)

#### Fall Detection Procedure



The wearer falls, the MMPLS registers a fall and a chime will sound


Sends an SMS to the primary contact and includes a google maps link. Also sends an alert to the mobile APP

Call cycle begins. First SOS contact # is called. If call is unanswered, the call moves to the next SOS contact # in sequence of order within 12 secs

#### NOTE

- If a false alert is detected, the wearer can cancel the alert by quickly pressing the SOS button.

#### GEO Fencing Procedure

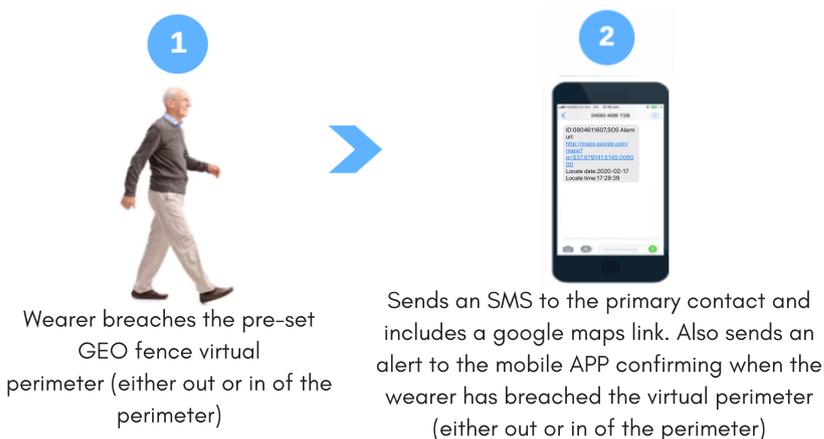

Wearer breaches the pre-set GEO fence virtual perimeter (either out or in of the perimeter)

Sends an SMS to the primary contact and includes a google maps link. Also sends an alert to the mobile APP confirming when the wearer has breached the virtual perimeter (either out or in of the perimeter)

#### NOTE

- The GEO fence perimeter can be set at a minimum of 200m from the point location. i.e. 200m from front gate of house
- If the GEO fencing feature has been requested upon purchase, please contact MindMe on 1300 776 907 to assist with initial setup.

# QUICK USER GUIDE - MindMe Pro LS

## Starter Guide

### Canceling a false alert

In circumstances where the SOS or fall alert has been accidentally activated, press the 'On / Off' button to cancel the alarm.



On/Off  
button

### Checking the MMPLS is turned on

Tap the 'SOS button' for 1 sec. If the device is turned on, the blue LED light will flash.



SOS  
button

### Getting a GPS reading



Prior to using your device you will need to get an initial fix for GPS, turn on the MMPLS and either take the MMPLS outdoors or place it near a window so the MMPLS can get a fix on the satellites. This could take up to 10 minutes.

#### NOTE

- If this is not completed prior to use, you may not get an accurate GPS location.

## MMPLS Mobile APP

### Downloading & Log into MMPLS Mobile APP

To monitor and program the MMPLS, you will need to download and log into the mobile APP (AIBEILE Plus). Your username and login details are detailed on the setup sheet provided.


1



AIBEILE Plus  
Lifestyle



2

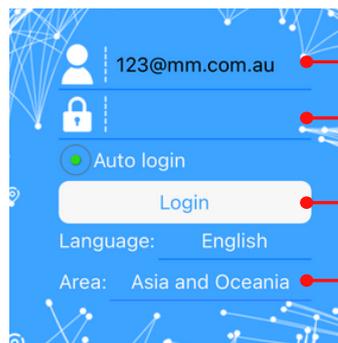

Login ID

Password

Login

Area

#### NOTE

- Abeile Plus is designed for the emergency contacts to monitor and change device settings, the wearer is **NOT** required to download Aibeile Plus to operate the MMPLS.

Download the Aibeile Plus mobile APP from the Apple Store or Google Play

Open the Aibeile Plus app and follow the instructions below:

- Enter the 'Login ID' and 'Password' which are detailed on the 'Setup and Overview sheet' provided (**PLEASE DO NOT use your own email address to login**)
- Ensure that 'Asia and Oceania' is selected in the 'Area' section
- Press 'Login', you will automatically be redirected to the homepage which is a map of the devices location

# QUICK USER GUIDE - MindMe Pro LS

## Starter Guide

### Mobile APP home screen overview

Below is an overview of some commonly used Aibeile Plus features and settings to assist in managing the MMPLS

#### NOTE


- To change any settings or features, the MMPLS will need to be turned on

**Envelop icon:** Press this icon to view a list of previous alerts that have been activated by the MMPLS. Alert type, time and date will appear.

**Address bar:** Displays the address where the GPS satellites last registered the MMPLS. The GPS feature will automatically update every 10 mins.

**Map & locator icon:** A map of the MMPLS location will appear. It will detail the MMPLS current battery level and the time and date that the MMPLS last connected with the GPS satellites.

**History:** Press the icon track the travel history of the watch.

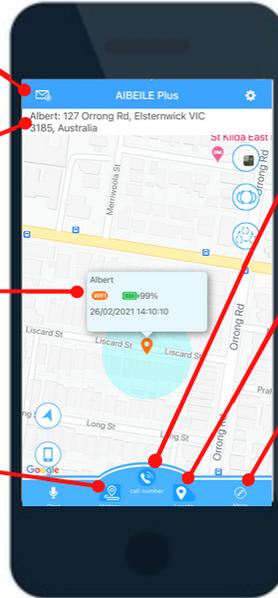


**Call number:** Press the icon to make standard phone call to the MMPLS. Once answered, a 2 way conversation can take place between the caller and wearer.

**Locate:** Press the icon to update the current location of MMPLS.

**More:** Press the icon to access features including:

- Settings:** Allows various settings/features to be programmed including SOS #'s, fall detection & phone book
- GEO fence:** Displays a pre-programmed or add a GEO fence
- Medicine notifications:** Displays a pre-programmed or set up of medication reminders


### Changing Emergency Contact #'s

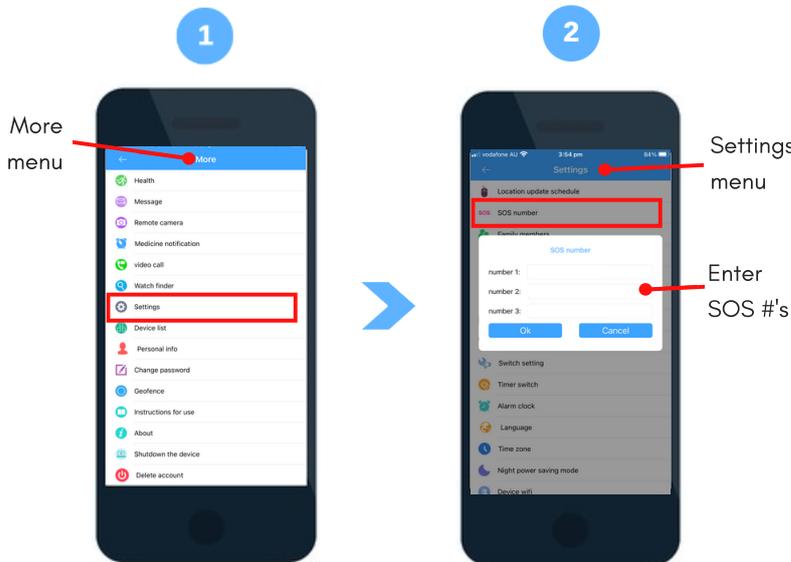

Select the 'More' icon from the homepage, then select 'Settings' from the options list.

From the 'Settings' menu, select the 'SOS number' option. A pop up box will appear, add or remove emergency contact #'s as desired and select 'Ok'. A grey pop up box displaying 'Set Ok' will appear if the changes have been successful.

### Changing fall sensitivity

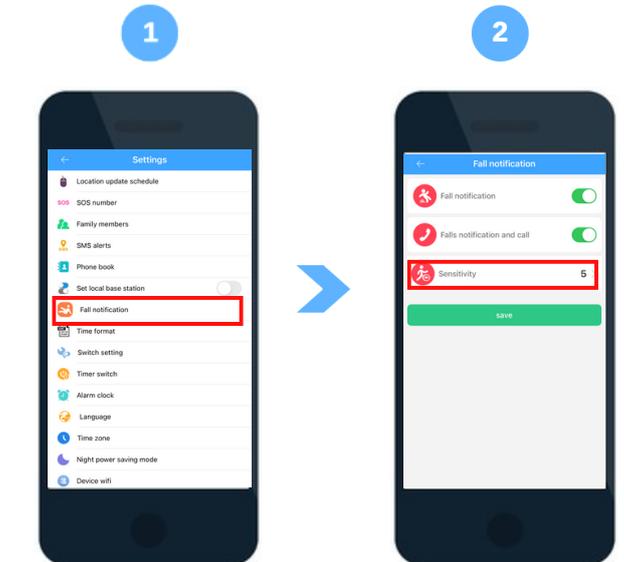

From the 'Settings' menu, select the 'fall notification' option.

Select the 'Sensitivity' option. Then select the desired sensitivity from the drop down box (1 = most, 6 = least sensitive, level 5 is recommended), press 'Ok' and 'Save'. A grey pop up box displaying 'Set Ok' will appear if the changes have been successful.